Quick Pick.

Pick. Seal. Submit

Diasorin

LIAISON® Q.S.E.T. Quick Pick

EN: For more information,
scan the code

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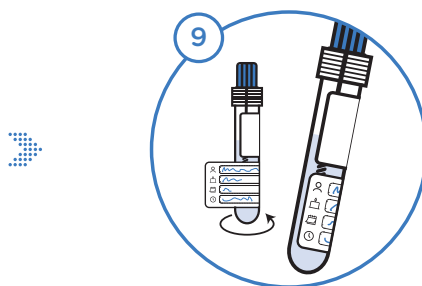
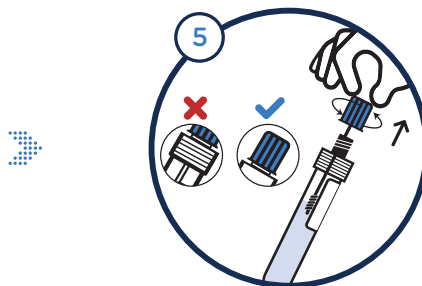
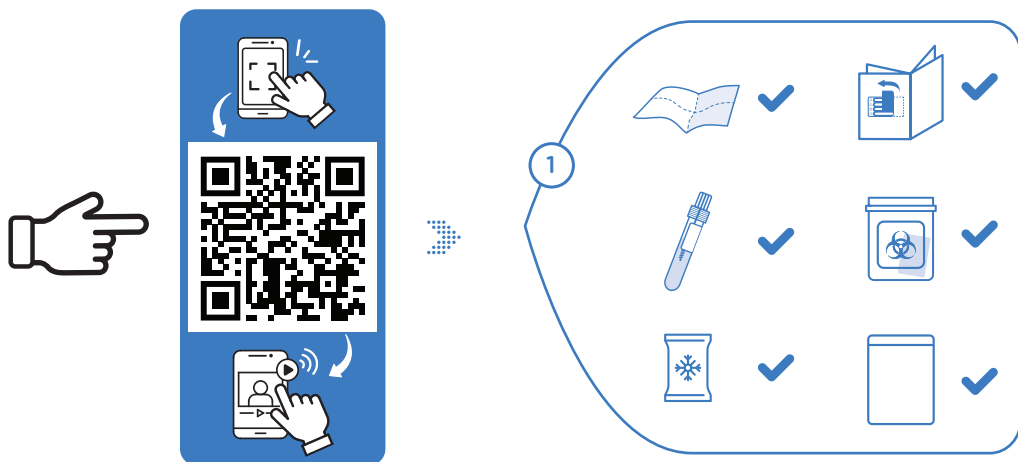
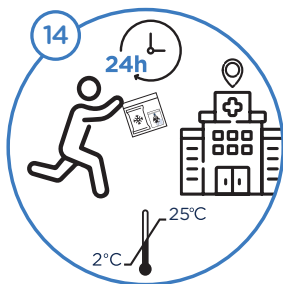
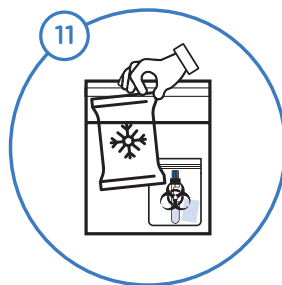
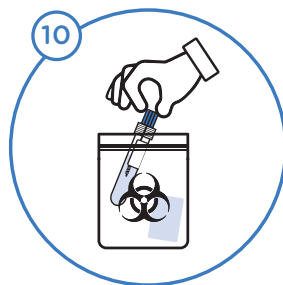
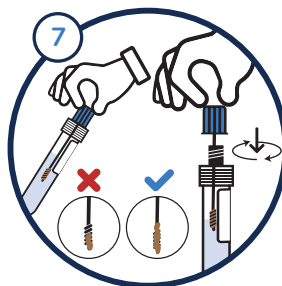
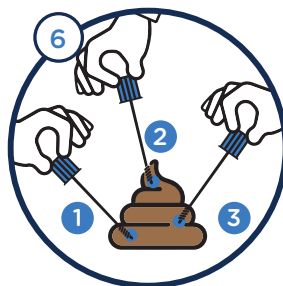
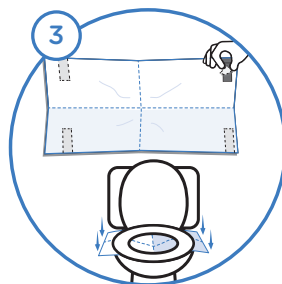
www.qset-quickpick.com

Read this instruction carefully and follow these steps to collect your stool. Children between the ages of 2 and 13 must be assisted by an adult for stool collection. For more information scan the QR code or visit www.qset-quickpick.com to watch a short demo video. **Return sample within 24 hours of collection. Don't collect if you can't meet this timeframe.**

1. Verify all components as illustrated in pictogram number 1 before sample collection. If any are missing, contact your healthcare provider.
2. Place **only** the provided ice pack in the freezer for **at least 10 hours before sample collection**.
3. Use adhesive strips to secure the stool paper under the seat and over the toilet bowl.
4. Sit on the toilet and collect stool on the paper. Avoid urine. **If stool is liquid or bloody, contact your healthcare provider.** Once the bowel movement is complete and stool is on the collection paper, stand up.
5. Unscrew **only** the blue cap of the sample tube and remove the sampling wand. Do **NOT** unscrew the clear cap.
6. Collect by touching the sampling wand to 3-5 different areas of the stool in sequence. Do **NOT** return the sampling wand to the tube between stool collections. Stool collection from a diaper is an acceptable alternative following the provided instructions.
7. Ensure the grooves are fully coated with stool. Insert the sampling wand back through the opening of the tube and tightly screw into place. Excess stool will be removed from the wand by the inner funnel.
8. Find the label on the back cover of the leaflet, fill it in clearly, remove it.
9. Attach the completed label to the bottom part of the sample tube, making sure not to cover the existing label.
10. Put the sample tube in the biohazard bag and do **NOT** remove the absorbent pad. Ensure the bag is tightly sealed.
11. Place the biohazard bag containing the sample tube and the absorbent pad together with the frozen ice pack inside the insulated pouch.
12. Peel off the liner and use the adhesive strip to seal the insulated pouch.
13. The stool collection paper may now be carefully removed from the toilet bowl. Throw it into the toilet and wait 5 minutes. Flush down the eco-friendly biodegradable collection paper.
14. Stool collection completed! The collection site must receive your sample within 24 hours of collection. Store the collected sample at 2-25 °C (35-77 °F), away from heat and direct sunlight. Do not freeze after the collection. Keep the collected sample out of reach of children.

WARNINGS

Do **NOT** drink the liquid inside the sample tube.

Do **NOT** let the liquid touch your skin or your eyes. If the liquid comes in contact with your skin or eyes, rinse thoroughly with water. Use of gloves is recommended.

Do **NOT** insert the sample tube or any part of it into your body.

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**DiaSorin Inc.**